Deutsch

Deutsch

MaxiFresh™

Gebrauchsinformation

MaxiFresh™

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MaxiFresh™

Fields of application:

The MaxiFresh™ cleaning tablet is used for the outer cleaning and care of dental splints and sport mouthguards.

Frequency of use:

We recommend daily cleaning. This will prolong the durability of your top-quality dental splint and safeguard an impeccable hygiene and function.

Composition:

The cleaning tablet contains potassium monopersulphate.

Application:

Put the cleaning tablet into the MaxiFresh™ bowl filled with 150 ml of water with a temperature of 37°C.

Please note: In cooler water the cleaning tablet might not dilute completely.

After the dilution of the tablet, the reaction time should be at least 15 minutes, the maximum should be 60 minutes. When reaction time has passed take the dental splint out of the MaxiFresh™ bowl, and – with the help of the insert – rinse thoroughly with clear water. Then dry the surface.

Child protection:

Make sure that this care product is stored away from children's reach.

Storage conditions:

At room temperature, i.e. 18°C-28°C

Durability: see date of expiry

MaxiFresh™ is not to be used when expiry date has passed.

Form of delivery:
item no. D-4100

24 MaxiFresh™
cleaning tablets incl.
MaxiFresh cleaning bowl

item no. D-4101

24 MaxiFresh™
cleaning tablets Refill Set

Code letter and hazard designation of product: ☒

Hazard-determining components of labelling:
pentapotassium bis(peroxymonosulphate)bis(sulphate)

Risk phrases:

22 Harmful if swallowed.
36/37/38 Irritating to eyes, respiratory system and skin.

Safety phrases:

1/2 Keep locked up and out of the reach of children.
13 Keep away from food, drink and animal feedingstuffs.
26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
36/37/39 Wear suitable protective clothing, gloves and eye/face protection.
45 In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).
56 Dispose of this material and its container to hazardous or special waste collection point.

This product was developed for dental lab use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. The user is generally obligated to check the product before use. Especially if the purpose is not mentioned in this instructions.

Date of information: July 2012

B&D Design 120711-8000-3833