## WELLSABITE HARD A85 / HEAVY D45 BISSREGISTRAT

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## WELLSABITE HARD A85 / HEAVY D45 BITE REGISTRATION

### INSTRUCTIONS

#### BITE REGISTRATION MATERIAL ON A POLYVINYLSILOXANE BASIS, ADDITION-CURING

All bite registration materials are manufactured from high-quality raw materials, and offer a wide range of applications due to their different final hardness. The materials are highly thixotropic and easy to apply with the automated safety cartridge system. They can be easily removed from the mouth, and provide long-term dimensional stability. The pleasantly mild mint flavour makes these materials particularly patient-friendly.

#### INDICATION:

D45 / A85: Produce long lasting, multiple-use bite records; fixation of support pin and facebow records.

#### PROCESSING:

Insert cartridge in 1:1 mixing gun, remove cap. When using a new cartridge, press out a small amount of material until a smooth flow is achieved. Screw mixing tip (green) onto cartridge and dispense desired amount of material. The used mixing tip remains on the cartridge. Before using again, remove the used mixing tip and check the discharge openings. Clean if necessary, then attach a new mixing tip and continue

as normal. D45 and A85 are millable and breakable due to their high final hardness.

#### SPECIAL INSTRUCTIONS:

- Some solutions used prior to recording, such as mouthwash solutions, may interfere with the setting process; any residues must therefore be thoroughly removed.
- Do not leave any residues of the recording materials in the oral cavity.
- The fully automated mixing rules out the negative effects of the material through contact with certain glove types, e.g. latex gloves. This sensitivity exists with all addition-curing silicones; it must be taken into account during processing.
- Cartridge materials should be used at room temperature (64.4°F - 77°F, or 18°C - 25°C) only.
- Disinfection: Impressions can be disinfected with 2% glutardialdehyde solution.
- Storage: between 41°F and 81°F (5°C - 27°C)  
Shelf life: see information on package

#### OPERATING AND SAFETY INSTRUCTIONS:

Avoid contact with eyes; if necessary wash eyes immediately with plenty of water, and consult an eye specialist without delay. Avoid contact with clothes, as curing silicones are chemically stable and cause smudges that cannot be removed. The processing, usage and application of the products delivered by us fall exclusively in the domain of your responsibility, as it is not possible for us to exercise any control in this regard. To the extent that we offer oral or written advice or provide consultancy through technical application-related experiments, any such consultancy is to be regarded as non-mandatory instructions only, even with regard to possible industrial property rights

of third parties. This does not release you from the obligation of testing the delivered products on your own, to ensure that they meet the intended objectives and processes. Our liability is limited in all cases and all instances of damage to the value of the products delivered by us and used by you. The faultless quality of our products is guaranteed according to the standard of our general business conditions.

Please note the warning in the safety datasheet!  
Info at [www.wellsamed.de](http://www.wellsamed.de)  
Errors and changes reserved.

#### TECHNICAL DATA AND PRODUCT PROPERTIES

\*Time specifications given in the table relate to 73.4°F (23°C) and a relative humidity of 50 +/- 10%. As a general rule, higher temperatures will result in shorter setting times and lower temperatures in longer setting times.

TECHNICAL DATA	WELLSABITE HEAVY D45 BISSGRGISTRATION	WELLSABITE HARD A85 BISSGRGISTRATION
Colour	turquoise	yellow
Dosage (1:1)	automatic	automatic
Working time incl. Mixing time*	30 sec.	30 sec.
Time in mouth	30 sec.	1 min.
Total setting time	1 min.	1 min. 30 sec.
Hardness	D45	A85
Linear dimensional change	0,1 %	0,1 %

Issue Date  
01/2013

wellsamed GmbH & Co. KG  
PF 10 13 08  
04013 Leipzig • Germany  
[www.wellsamed.de](http://www.wellsamed.de)