GLUMA® Solid Bond

(DE)	Gebrauchsanweisung	2
(GB)	Instructions for use	5
(FR)	Mode d'emploi	8
(ES)	Instrucciones de uso.....	11
(IT)	Istruzioni per l'uso	14
(PT)	Instruções de uso	17
(NL)	Gebruiksaanwijzing	20
(SE)	Bruksanvisning	23
(DK)	Brugervejledning	26
(NO)	Bruksanvisning	29
(FI)	Käyttöohjeet.....	32
(GR)	Οδηγίες χρήσης	35

Heraeus

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GLUMA® Solid Bond

Instructions for use 

GLUMA Solid Bond is a bonding agent system for the adhesive filling therapy consisting of GLUMA Solid Bond P and GLUMA Solid Bond S and is used in combination with an etching agent (e.g. iBOND® Etch 35 Gel). GLUMA Solid Bond P is a dentine-primer, whereas GLUMA Solid Bond S is an X-ray opaque sealer yielding fluoride ions.

Attention: In order to perform a complete sealing of cavities GLUMA Solid Bond P as well as GLUMA Solid Bond S both have to be applied.

Composition:

GLUMA Solid Bond P contains ethanol, (2-hydroxyethyl-)methacrylate, maleic acid, TEGDMA
GLUMA Solid Bond S contains bisphenol A-diglycidildimethacrylate, TEGDMA

Application:

- for the adhesive improvement of: composite-filling materials
sealers at enamel and dentine
- to cover the cavity lining
- to seal exposed cervical areas

Application steps:

To prepare the treatment it is recommended to clean the hard dental substance with a polishing paste which is free of fluoride.

1. Preparation of the cavity according to the general regulations of the composite-filling therapy. Then clean and dry the cavity. For deep cavities it is recommended to protect the dentin adjacent to the pulp by a corresponding lining (e.g. calcium hydrate preparation and filling of glass-ionomeric cementum). For the sealing of exposed cervical areas a preparation normally is not necessary. It is recommended to use rubber dam.
2. Apply etching agent (e.g. iBOND® Etch 35 Gel) to the whole cavity starting from the declined enamel areas and leave it for 15–30 sec (total-etch-technique). If desired apply etching agent only to the enamel. In case of fluoridated enamel it may be necessary to repeat steps 2–4. Please refer to the instruction for use of the appropriate etching agent.
3. Completely rinse off etching agent with water for at least 15 sec by using water spray. No rests should remain in the cavity.
4. Dry cavity with oil-free air (Do not desiccate dentin!).
5. Place GLUMA Solid Bond P and massage it smoothly in the dentin. The adhesion of enamel is not affected by GLUMA Solid Bond P.
6. Blow by means of a low airstream.
7. Place GLUMA Solid Bond S and apply to the cavities as well as the etched enamel areas.
8. Spread it by means of a low airstream. Do not blow layer too thin! In case of deep cavities apply twice if required.
9. Light cure for 40 sec (with Translux®-light lamp). Do not touch or remove dispersion layer.
10. Apply composite filling material (e.g. Venus® Diamond, Venus® Pearl, DURAFILL® VS) according to producer's specifications.

After polishing or sealing of the composite-filling it is recommended to perform a fluoridation of the treatment area.

Further precautions:

In case of direct contact with dentin which is adjacent to the pulp irritations may not be excluded. Thus dentin adjacent to the pulp shall be covered with a calcium hydrate preparation as well as a lining (e.g. glass-ionomeric cement).

Do not use together with preparations containing eugenol as they may impede the hardening process.

GLUMA Solid Bond S: Irritates eyes and skin. Sensitization may be caused by skin contact.

GLUMA Solid Bond P: Inflammable! Irritates eyes and skin. Sensitization may be caused by skin contact.

Therefore, avoid the contact with skin and eyes. In case of contact with the material rinse with plenty of water and seek medical advice if necessary. For use by dental personnel only.

Store GLUMA Solid Bond P in the refrigerator at 4°C (39°F). (Intermediate storage up to 2 months at 25°C (77°F) is possible.) The material should not be used after the expiry date. Keep out of direct sunlight.

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Heraeus Kulzer GmbH
Heraeus Kulzer, LLC