Prior to use, carefully read the instructions for use.

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GC Fuji I CAPSULE

RADIOPAQUE GLASS IONOMER LUTING CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- 1. Cementation of metal-based inlays, onlays, crowns, bridges and posts
2. Cementation of high strength (zirconia based) all ceramic crowns and bridges


CONTRAINDICATIONS

- 1. Pulp capping.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

Table with 2 columns: Powder / Liquid ratio (g/g), Mixing time (sec.), Working time (min., sec.) (at 23 °C), Final finishing commencing time (after seating the restoration)

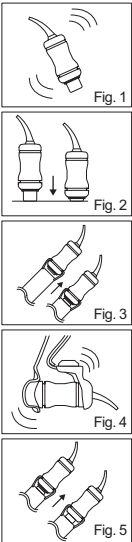- 1. TOOTH PREPARATION
a) Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.
b) Clean the prepared teeth with pumice and water.
c) Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air.
2. RESTORATION PREPARATION
Make sure that the restoration is pretreated and handled according to the manufacturer's instructions.
3. CAPSULE ACTIVATION AND MIXING
a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder.
b) To activate the capsule push the plunger until it is flush with the main body.
c) Immediately place the capsule into a GC CAPSULE APPLIERS and click the lever once.
Note: The capsule should be activated just before mixing and used immediately.
d) Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/-4,000 RPM).
Note: When using a rotational mixing device, it is recommended not to use the centrifuge option.

- 4. DISPENSING
a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
b) Make two clicks to prime the capsule.
c) Extrude directly into the restoration.
d) To remove the used capsule, push the applier release button.
5. CEMENTATION
a) Coat the internal surface of the restoration with sufficient cement and seat immediately.
b) Maintain moderate pressure.
c) Start removing excess cement when the excess cement feels rubbery.
d) Finishing can be started 4 minutes 30 seconds after seating the restoration.

STORAGE

Store the capsules in the original aluminium foil in a cool and dark place (4-25°C) (39.2-77.0°F). (Shelf life : 2 years from date of manufacture).

SHADE

Light yellow.

PACKAGE

50 capsules
Average contents per capsule : 0.33g powder and 0.18g (0.14mL) liquid.
Minimum net volume of mixed cement per capsule : 0.19mL.
Option : GC CAPSULE APPLIER (1 piece).

CAUTION

- 1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. This product is not indicated for filling or core build-up.

Last revised : 04/2008

MANUFACTURED BY GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

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