## F

### COMPOSITION

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### PROPERTIES

Epoxy resin- and calcium hydroxide-based root-canal sealer, designed for permanent root-canal filling. ACROSEAL is eugenol-free.

### INDICATIONS

Root canal sealer, to be used in conjunction with gutta-percha points.

### CONTRAINDICATIONS

Hypersensitivity to any component of the formulation.

### INSTRUCTIONS FOR USE

Isolate the tooth to be filled.

• **Tubes**:

Dispense two equal lengths of BASE and CATALYST from both tubes, on a mixing pad. Spatulate the two pastes until a homogeneous consistency is obtained.

• **“Automix” syringe**:

Attach a mixing tip to the double-syringe and push out the required amount of paste onto a mixing pad.

Coat the gutta percha point(s) with ACROSEAL and condense the point(s).

Take X-rays to check the quality of the root filling.

#### SPECIAL WARNINGS AND PRECAUTIONS FOR USE

As with any root canal filling material, only a perfectly shaped and disinfected canal can be filled. Determine the working length of the canals before filling, to prevent apical extrusion.

#### *Special risks*

**BASE**: Harmful. May cause sensitisation by inhalation and skin contact.

**CATALYST**: Contains epoxy compounds.

Corrosive. Causes burns. May cause sensitisation by skin contact.

Dangerous for the environment. Toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

#### *Safety advice*

Avoid contact with skin. Wear suitable protective clothing, gloves and eye/face protection. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). Avoid release to the environment. Refer to special instructions/safety data sheets.

#### REMARKS

- When opening the tube of BASE, squeeze out the supernatant liquid layer if present.
- The setting time of ACROSEAL, in vivo, ranges from 16 and 24 hours and varies according to hygrometry.
- Spatulas, mixing pads or any other instrument should be cleaned with alcohol or acetone, immediately after use.

#### STORAGE

Keep in a cool place (between 10 and 25°C)

Tightly close the tubes after each use.

#### PRESENTATION

Box containing:

- 1 x 8.5 g tube of BASE
- 1 x 9.5 g tube of CATALYST
- 1 cross-ruled mixing pad

Box containing:

- 2 x 8.6 g "Automix" syringes
- 15 mixing tips
- 15 intraoral tips
- 1 mixing pad

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