Prior to use, carefully read the instructions for use.

EN

FREEGENOL TEMPORARY PACK

EUGENOL FREE TEMPORARY CEMENT

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Cementation of temporary crowns and bridges.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad.
- Mix thoroughly for 20-30 seconds.
- Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.
- Allow to set (3 minutes from start of mixing) and trim excess.

Note:

- Setting time can be adjusted by using more or less Base (More Base-faster set ; less Base-slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).
- If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life: 3 years from date of manufacture)

PACKAGES

1-1 pkg : 55g Base, 20g Accelerator, Cleaner and mixing pad.

CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 09/2008

CE 0086

GC

MANUFACTURED by
GC DENTAL PRODUCTS CORP.

2-285 Toriimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32. 16. 39. 80. 50.

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900.

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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