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Harvard TEMP Cem – Harvardmix-Syringe

GB

Instructions for use for the dentist

Areas of application

Non-eugenol temporary crown and bridge cement suitable for cementing Temporary crowns or bridges as well as for trial cementing of permanent restorations.

Composition

Pastes with zinc oxide and organic acids.

Contra indication

- Do not use Harvard TEMP Cem with patients who show an allergy to ingredients of the material.

Special information

- Do not dry stumps, best moisten them with some water.
- A humid stump additionally facilitates to remove the Temporary restoration later on.

Warning advice

- In case of skin or eye contact, immediately rinse with plenty of water.
- If swallowed seek medical advice.

Processing

1. Preparing the Harvardmix-syringe

Remove the cap of the Harvardmix-syringe and throw it away (**do not use it again!**). It is replaced by the 1:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application from the mixing cannula. Discard the first 2-3 mm of the extruded material. This must be done for each new mix.
Store used syringe with fixed used mixing cannula.

2. Cementing

Harvard TEMP Cem can be applied directly into the crown or bridge under slight pressure from the mixing cannula. Use a thin layer to the inside of the crown or bridge and seat not later than 1:30 minutes (working temperature (23 °C, 73 °F) cautiously but firmly over the prepared tooth structures.
Harvard TEMP Cem sets in the mouth within 2-3 minutes. After it has set remove any excess.

Packages

- Harvard TEMP Cem – Harvardmix-syringe 5 ml, 10 mixing cannulas
- Harvard TEMP Cem – Harvardmix-syringe 10 ml, 20 mixing cannulas
- Mixing cannulas for Harvard TEMP Cem – Bag with 10 pieces

Harvard TEMP Cem – Harvardmix-syringe

- Store at 8 - 25 °C (46 - 77 °F)!
- Do not use after the expiration date!
- Corresponds to the standard DIN EN ISO 3107 and ADA/ANSI-Specification No. 30.