Panasil® Haftlack

37034/5111

Gebrauchsinformation	Deutsch	2
Directions for use	English.....	3
Mode d'emploi	Français	4
Istruzioni per l'uso	Italiano	5
Instrucciones de uso	Español.....	6
Instruções de uso	Português	7
Bruksanvisning	Norsk	8
Brugsinformation	Dansk	9
Bruksanvisning	Svenska.....	10
Kullanım Kılavuzu	Türkçe.....	11
Gebruiksaanformatie	Nederlands.....	12

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Panasil® Adhesive

Adhesive for Kettenbach addition-curing silicones.

Distributed in the U.S. by: **Kettenbach LP**
 400 Oser Ave., Suite 1650
 Hauppauge, NY 11788
 USA

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

Made in Germany
 37034/5111

Product Description/Indication

Adhesive for Kettenbach addition-curing impression and reline impression materials. Adheres to metals and acrylics.

Instructions for use

Apply a thin coat of Panasil® Adhesive onto the impression tray and allow it to dry for approx. 5 minutes. Warmth and compressed air curtail the drying time. When using plastic impression trays, please check that they are compatible with the adhesive. Replace the cap on the bottle immediately after use!

After mixing the impression material, load it into the tray within 30 minutes.

Remains of impression material can be cleaned off the tray with cleaning solvent, petroleum ether or orange oil.

Warnings

Do not use Panasil® Adhesive with condensation-curing silicones, polyether or polysulfide materials.

Precautions

Do not use after the expiration date.

Do not swallow the material! If accidentally swallowed: consult a doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Avoid contact with skin.

Do not allow the material to enter the sewer or water system to avoid environmental hazard. Do not dispose of with household waste.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Keep away from sources of ignition.

Side effects

Although there are no known systemic side effects, specific contact allergies cannot be ruled out in rare cases.

Do not use the material in case of known allergy to one of the ingredients.

Composition

Silicone resin, solvent (ethyl acetate, xylene).

Storage Conditions:

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

- R 11 Highly flammable.
- R 20/21 Harmful by inhalation and in contact with skin.
- R 36 Irritating to eyes.
- R 66 Repeated exposure may cause skin dryness or cracking.
- R 67 Vapors may cause drowsiness and dizziness.

Trademarks

Kettenbach and Panasil® Adhesive are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Not all Kettenbach products are available in all countries.

Legend

Conformity with MDD 93/42/EEC		Use by	
Temperature limitation		Keep dry	
Keep away from sunlight		Catalog number	REF
Batch code		Caution, consult accompanying documents	
Dual system (Germany only)		Rx only For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.	
Highly flammable		Irritant	

Limitation of liability

Except where prohibited by law, Kettenbach GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

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Panasil® Adhesief is gedeponeerde handelsmerken van Kettenbach GmbH & Co. KG.