Prophylaxekanüle Prophylaxis cannula




CE 0124









9000-616-26

 DÜRR
DENTAL

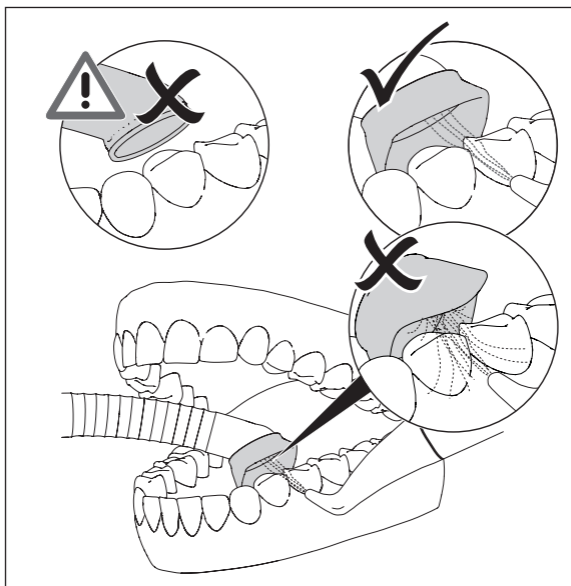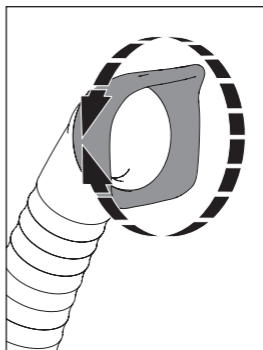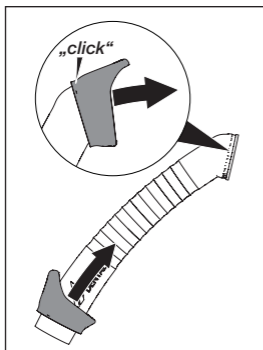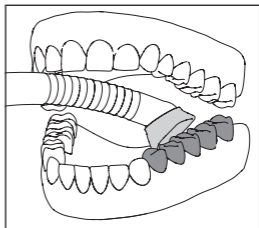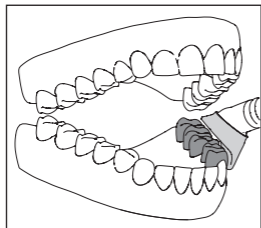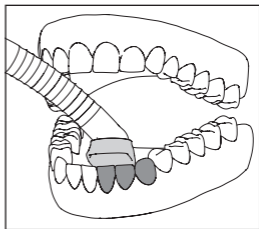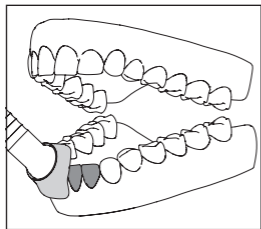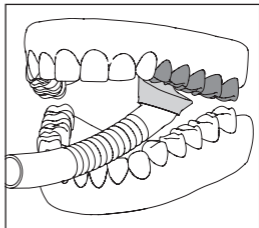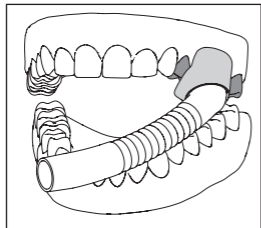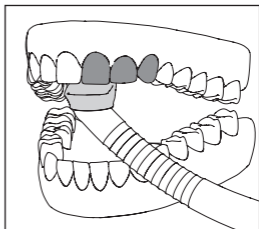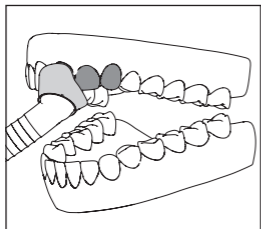2011/12

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The prophylaxis cannula is used for the suction of secretion and spray mist from the patient's mouth (e.g. during tooth cleaning with air abrasion systems).

Operation

see page 2 and 3.

Before the first use, clean, disinfect and sterilise the prophylaxis cannula.

Preparation

Cannula are medical instruments which come into contact with mucous membranes and skin or tissue which has been affected by illness.

Remember to observe all current local and national regulations concerning the cleaning, disinfection, sterilisation and storage of cannula.

When preparing the cannula do not forget to wear personal protective equipment.

According to the Recommendations of the Commission on Hospital Hygiene and the Prevention of Infections, in the Federal Republic of Germany this medical product is categorised in group: "Semicritical Group B with increased requirements for preparation and use".

The instructions described below have been validated by Dürr Dental as suitable and are recommended for the preparation of cannula before and after use. The surgery manager is responsible for ensuring that the correct and safe preparation of the instruments is carried out using suitable appliances, materials and trained personnel in order to produce the desired results. Validation and routine monitoring of the preparation procedures are necessary. Any and every deviation from the procedures described below by preparation staff is solely the responsibility of the personnel in terms of effectivity and the possible negative repercussions.

Transport / Storage

- After use, place cannula in a suitable closed container which is filled with an instrument disinfectant solution (e.g. ID 212, ID 212 forte, ID 213; observe the manufacturer's instructions) collect and transport in a securely sealed vessel to the site of cleaning.

Preparation for cleaning

- Take off the rotating head backwards.
- Where necessary, remove larger dirt particles etc. using a disinfectant disposable wipe (e.g. FD 350).

Cleaning and disinfection, automatic

Use cleaning and disinfection devices strictly according to manufacturer's instructions.

- Place the disassembled cannula into the unit in such a way that the interior surfaces will be rinsed thoroughly and the water can drain off.
- Set up the cleaning and disinfection cycles according to the manufacturer's instructions.
- After the cleaning and disinfection cycles have finished check the cannula carefully for signs of residual dirt and repeat the cycles if necessary.

Cleaning and disinfection, manual

Cleaning takes place in a wet chemical process using a VAH/DGHM listed instrument disinfection agent:

- Place the dismantled cannula fully submersed in an instrument-disinfectant and cleaning agent (e.g. Dürr ID 212, ID 212 forte; ID 213, observe manufacturer's instructions carefully).
- After the cleaning and disinfection cycles have finished check the cannula carefully for signs of residual dirt, if necessary clean with a suitable brush and repeat the cycle.
- Rinse cannula thoroughly with water.
- Drying should be carried out using hygienically clean, disposable wipes, or under the application of compressed air.

Check and test of function

Frequent application and cleaning has little effect on cannula. The end of the product's life cycle is determined by the amount of wear and tear or damage during usage.

- Check the cannula visually and where necessary, damaged cannula must be disposed of. Before disposal cannula must be disinfected and sterilised.

Packaging

- Sterilise disassembled cannula either unpacked or packed in a container or in sterile product packaging. Use a suitable sterile product packaging!

Steam Sterilisation

WARNING

The wrong sterilisation can reduce effectiveness and can lead to damage to the product

- Only steam sterilisation is permissible.
- Do not exceed 134 °C.
- Observe the process parameters.
- Do not use any other process.
- Place filter paper on sterilisation tray before use.
- Always place objects for sterilisation together with instruments in order to achieve a balanced temperature distribution during sterilisation.
- Avoid metal contact between any of the objects for sterilisation and any instruments or with the tray edge itself.
- Place the objects to be sterilised in such a way that they do not come too close to the heating elements and do not come into contact with the internal walls and place on the middle tray.
- Only sterilise the product in a disassembled state.

Temperature: 134 °C

Gauge pressure above ambient environment: 2.16 bar / 0.216 MPa

Holding time: 5 min

- Sterilised implements must be sterilised in a small steam sterilisor before use according to European guidelines EN 13060 in sterilisation cycle B and subsequently dried.
- Observe the manufacturer's instructions concerning usage of the steam sterilisation unit and the correct placement of objects to be sterilised.
- Observe all national and local guidelines and regulations concerning sterilisation of medical products.

Storage

Store the cannula so they are protected against any possible contamination.

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DÜRR DENTAL AG