# Xantopren® function

Heraeus Kulzer GmbH  
Grüner Weg 11  
63450 Hanau (Germany)  
info.dent@heraeus.com  
www.heraeus-dental.com  
Importado e Distribuído por Heraeus Kulzer South America Ltda.  
CNPJ 48.708.010/0001-02  
Av. Marques de São Vicente, 2800 – São Paulo – SP – CEP 05036-040  
sachverständiger: \_\_\_\_\_  
Responsible: Dra. Regiane Marton – CRM 70.705  
Nº ANVISA: vide embalagem

66040260/01 – D.O.G./09.09

Heraeus

## Xantopren® function

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Stand: 09.2009

## Xantopren® function

Special silicone impression material for functional impression taking.

Indications for use:

- Closed mouth functional impression with Xantopren® function Activators No. 1 Plus and 2 Plus
- Open mouth functional impression with Activator universal Plus

Precautions for handling: Visually inspect for damage before first use. Damaged products must not be used. Close carefully after use. Please use only the accessories included with the package (spoon, dropper). Since all silicone impression materials will stain clothing or surfaces once cured, contact should be avoided.

Procedure:

Mixing Ratio: 1 spoon of Xantopren® function (0,5 ml/14,4 g) with

A: 4 drops Xantopren® function Activator No. 1 Plus (red) (0,08 ml/0,09 g) and 4 drops Xantopren® function Activator No. 2 Plus (light green) (0,11 ml/0,10 g) or

B: 6 drops Xantopren® function Activator No. 1 Plus (red) (0,13 ml/0,13 g) and 6 drops Xantopren® function Activator No. 2 Plus (light green) (0,17 ml/0,16 g) or

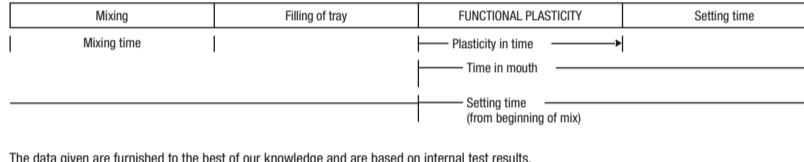
C: 3 drops Activator universal Plus Liquid (0,07 ml/0,07 g) or

D: 2 Teilelische Activator universal Plus Paste (0,10 ml/0,11 g)

Mixing: Flatten the Xantopren® function into a thin disc on the mixing pad. Using the spatula, carve shallow grooves into putty in a criss-cross pattern.

Add Xantopren® function Activator No. 1 Plus and No. 2 Plus according to sections A or B. Alternatively Activator universal Plus Liquid according to section C or Activator universal Plus Paste according to section D. Spatulate thoroughly for 30 to 60 seconds.

Impression Taking: Duration of plasticity phase in the mouth before setting (see diagram) is dependent on the Activator and the mixing ratios.

Times stated are related to room temperature of 23 °C/73 °F, 50% rel. humidity.

The data given are furnished to the best of our knowledge and are based on internal test results.

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