## Gebrauchsinformation

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For professional use only.  
Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

# Orthoskavident® C

## Composition

**Orthoskavident® C** contains: 2-Propanol, purest, DAB, Ph. Eur., PB, USP, Acetone, purest, Ph. Eur., BP, NF

## Range of indications

**Orthoskavident® C** is for cleaning and drying cavities and teeth after preparation thus permitting fillings, adhesives and other luting materials to bond properly.

**Contraindications:** None known.

**Side effects:** Causes serious eye irritation. May cause drowsiness or dizziness.

**Interactions with other medication:** None known.

## Mode of application

Soak a cotton pledget in **Orthoskavident® C** and use it to dab prepared cavities or preparations carefully to clean and dry them. Contact with larger areas of oral tissues causes dehydration and degreasing and must be avoided.

If you are using dental cauterization supplies, make sure that the previously applied Orthoskavident® C has dried completely to ensure that the excessive penetration of the tooth by the cauterization substance can be ruled out with certainty.

## Note:

To prevent contamination, place the quantity of Orthoskavident® C required into a mixing glass (or any other glass container) and obtain the material needed from this container. Dispose of any residues.

Once you have removed the quantity you need from the original container, please close this container immediately.

Storage conditions: Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

The user is responsible for the proper utilization of the product.

Do not use after expiration date.

The product does not normally cause allergic reactions. However, for sensitive persons, an allergic reaction to the product cannot be ruled out.

## Please note

Highly flammable liquid.

|                |  |
|----------------|--|
| H225           | Highly flammable liquid and vapour.  |
| H319           | Causes serious eye irritation.   |
| H336           | May cause drowsiness or dizziness.   |
| EUH066         | Repeated exposure may cause skin dryness or cracking.  |
| P210           | Keep away from heat/sparks/open flames/hot surfaces. - No smoking.   |
| P233           | Keep container tightly closed.   |
| P305+P351+P338 | IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. |

If Orthoskavident® C leaks out due to spillage or bottle breakage, dilute immediately with large quantities of water and wipe up. Ventilate the room well.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: 9th January 2012

Made in Germany  
37205/0312

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Kettenbach GmbH & Co. KG,