WELLSAWIPES

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REF WCW407

REF WCW409

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REF WCW409

Product description

WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER are ready to use alcohol-soaked disinfection wipes in a refill bag for application with a dispenser bucket. They are soaked with WELLSA-CLEAN S+ SCHNELLDESINFEKTION (containing alcohol and quarternary ammonium compounds) and can be used directly after insertion into the dispenser bucket. WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER provide an universal and time-saving application.

Areas of application

For rapid disinfection of alcohol-resistant surfaces of non-invasive, non-critical medical devices and medical equipment as well as all kinds of wipeable surfaces.

Suitable for e.g.:

- Surfaces close to patients, dental chairs, examination couches, operating tables
- Areas in medical and dental practices which are used for medical purposes
- Rapid disinfection between surgeries and treatments
- Clothes like OR shoes, OR aprons, X-ray aprons
- Alcohol-resistant surfaces of medical equipment
- Materials according to material compatibility (see page 2)

Not tested for food and veterinary areas. Not suitable for disinfection of skin and hands.

Not suitable for disinfection of invasive medical devices. Do not use for disinfection of semi-critical and critical medical devices.

Special Advice

Read safety datasheet and technical datasheet before use. Application only by qualified personnel.

Personal protection

Put on protective gloves, face masks and eye protection prior to disinfection! Provide adequate ventilation.

Application

WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER are applied for wipe disinfection. In case the surfaces are contaminated with organic material (e.g. blood, secretions etc.): Remove the visible contamination with a disinfection wipe and discard it. Subsequently, disinfect surface with another disinfection wipe. Wipe the surface applying mild pressure (wet wiping). Keep the surface completely moist during contact time. After contact time has elapsed: Remove residues with a clean, dry, disposable cloth or allow to dry.

WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER are intended for single use only. The declared efficacy is not ensured upon multiple use. Inaccessible parts have to be wetted with WELLSACLEAN S+ SCHNELLDESINFEKTION, kept wet during the required contact time and afterwards dried with an undyed disposable paper towel. A disinfection does not substitute the cleaning of the surface.

Summarized efficacy spectrum

Efficacy under dirty conditions (product ready to use, at room temperature)	Contact time
Bactericidal/Yeasticidal (VAH standard methods)	5 min
Bactericidal/Yeasticidal (EN13727, EN 13624, EN 16615)	5 min
Mycobactericidal/TB (EN 14348)	
Limited spectrum of virucidal activity (EN 14476, EN 16777)	
Virucidal activity against enveloped viruses (EN 14476)	30 sec

See page 2 for a detailed efficacy spectrum, material compatibility, and advice on filling of the dispenser bucket.

Further information on cleaning and disinfection of surfaces and medical devices:

Recommendation of the Commission for Hospital Hygiene and Infection Prevention at the Robert-Koch Institute (RKI) entitled „Hygiene Requirements for Cleaning and Disinfecting Surfaces“. Available online.

WELLSAWIPES

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REF WCW407

REF WCW409

Detailed spectrum of efficacy

Efficacy (product ready to use, at room temperature)	Standard	Contact time
BACTERIA		
S. aureus, E. hirae, P. aeruginosa, E. coli, MRSA → dirty conditions	EN 13727	30 sec
S. aureus, E. hirae, P. aeruginosa → dirty conditions	EN 16615	5 min
S. aureus, E. hirae, P. aeruginosa → dirty conditions	Standard methods VAH*	5 min
YEASTS		
C. albicans → dirty conditions	EN 13624	30 sec
C. albicans → dirty conditions	EN 16615	5 min
C. albicans → dirty conditions	Standard methods VAH*	5 min
MYCOBACTERIA/TB		
M. terrae, M. avium → dirty conditions	EN 14348	1 min
VIRUSES		
Enveloped viruses		
MVA Modified Vacciniavirus Ankara → dirty conditions	EN 14476	30 sec
BVDV → dirty conditions		
Non-enveloped viruses		
Adenovirus type 5 → dirty conditions	EN 14476	5 min
MNV murine Norovirus → dirty conditions		
Human Rotavirus → clean conditions		
MNV murine Norovirus → dirty conditions	EN 16777	
Adenovirus type 5 → dirty conditions		

*standard methods of the VAH (Association of Applied Hygiene) for surface disinfection by wiping (quantitative carrier test with mechanical action, practical conditions).

Material compatibility

Metals	aluminium, stainless steel, copper, brass
Plastics	PSU, PVC, silicone
Rubber	EPDM, latex
Advice	Upon compliance to the indicated contact time, no material damage is to be expected with the above mentioned materials. Avoid contact of longer than 48 hours with linoleum flooring and polycarbonate (Makroion). Avoid contact with acrylates. May cause stress corrosion on acrylic glass. Test material compatibility for polyacrylates, polycarbonates as well as not mentioned materials in an inconspicuous place before use.

Filling of the dispenser bucket

Prior to each initial filling/refilling, the dispenser bucket and its lid have to be cleaned thoroughly either by hand or in a dishwasher and then disinfected e.g. with WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER and/or WELLSACLEAN S+

SCHNELLEDESINFEKTION. Always wear a new pair of protective gloves when you disinfect and fill the dispenser bucket. Lot number, manufacturing and expiry dates provided on the refill bag are recorded on the additional label. The additional label is attached to the dispenser bucket and the refill bag is put into the dispenser bucket. The refill bag is cut open on its top and the first wipe from the middle of the roll is inserted through the opening of the lid. Then, the dispenser bucket is closed. The product is ready to use now. Close the lid properly every time a wipe is taken out to avoid drying of the wipes and formation of germs.

Advice

Avoid contact with other disinfectants. Do not mix with other disinfectants. Do not dilute. We recommend the application of WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER in the dispenser bucket available from wellsamed. We do not accept any liability for any other wipe-dispenser bucket combinations. Apply on smooth surfaces only. On coarse surfaces lint formation cannot be excluded. The WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER must not be disposed of in the toilet. Do not use after expiry date. The expiry date of the refill bag is indicated on the label.

Shelf life

Shelf life of the ready to use wipes in the dispenser bucket: 28 days from date of opening if the lid is closed properly and tightly every time a wipe is taken out.

Storage advice

Protect from heat and sunlight. Store in an upright position. Keep away from sources of ignition – No smoking. Keep away from food, drink, and animal feedingstuffs. Keep locked up and out of the reach of children. Keep container tightly closed and in a well-ventilated place.

See safety datasheet for further advice.

Composition

100 g of soaking solution contain:
33.66 g ethanol, 15.84 g 2-propanol, 0.05 g didecyltrimethylammonium chloride 70%, perfume*2 (not applicable for perfume-free)

Physico-chemical data

appearance:	clear and colourless
pH-value:	5.0 – 6.0
density (20 °C):	0.900 – 0.920
flash point (EN 22719):	22.75 + 0.5 °C

*2 Please note that perfume may cause allergies.

WELLSAWIPES

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REF WCW407

REF WCW409

Expert opinions & further information

WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER have been assessed by independent laboratories.

The listings, reports, and notes in this product information refer to the product PLIWA Quick & Clean S-Wipes, which is identical in formulation.

To exclude adverse effects on the skin upon unintentional contact with the product, a dermatologically survey has been carried out by the company Dermatest.

For further details concerning expert opinions, informations on products and answers to specific questions contact us by e-mail (info@wellsamed.de).

Identification, certificates

Medical device class II a / MD 0108

The PLIWA Quick & Clean S-Wipes, which has the same formulation as the WELLSAWIPES GETRÄNKTE DESINFEKTIONSTÜCHER, is VAH-listed.

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REF WCW103 3 litre bucket 70 sheets of fleece towel roll (REF WCW407)

REF WCW105 5 litre bucket 90 sheets of fleece towel roll (REF WCW409)

Labelling according to Regulation (EC) No 1272/2008 (CLP)

For information on labelling according to Regulation (EC) No 1272/2008 (CLP) see safety datasheet and label.

UN 3175

The manufacturer has a manufacturing authorisation according as per §13 German medicines act as well as a GMP- and an EN 13485 certificate.

 PLIWA Hygiene GmbH,
Pliwastraße 2, 34323 Malsfeld, Germany

Distributed by:
wellsamed GmbH
Brahestraße 16, 04347 Leipzig, Germany

Please report serious incidents related to this product to the manufacturer and the respective authorities.