Prior to use, carefully read the instructions for use. EN

## G-aenial

LIGHT-CURED RADIOPAQUE COMPOSITE RESTORATIVE  
For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**  
A. G-aenial ANTERIOR  
1. Direct restorative for Class III, IV, V cavities.  
2. Direct restorative for wedge-shaped defects and root surface cavities.  
B. G-aenial POSTERIOR  
1. Direct restorative for Class I and II cavities.

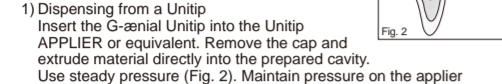
**CONTRAINDICATIONS**  
1. Full-capping.  
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.  
**DIRECTIONS FOR USE**  
1. Shade selection  
Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate G-aenial shades by referring to the G-aenial shade guide.  
2. Cavity preparation  
Prepare cavity using standard techniques. Dry by gentle blowing with oil free air.  
Note:  
For full-capping, use calcium hydroxide.  
3. Bonding Treatment  
For bonding G-aenial to enamel and / or dentin, use a light-cured bonding system such as GC G-aenial Bond, GC G-BOND™, GC Fuji BOND LC or GC UniFil® Bond (Fig. 1). Follow manufacturer's instructions.  
4. Placement of G-aenial  
1) Dispensing from a Unitip  
Insert the G-aenial Unitip into the Unitip APPLIER or equivalent. Remove the cap and extrude material directly into the prepared cavity.  
2) Dispensing from a syringe  
Remove syringe cap and dispense material onto a mixing pad. Place the material into the cavity using a suitable placement instrument. After dispensing, screw syringe plunger anticlockwise by a half to turn to release residual pressure inside the syringe. Replace cap immediately after use.  
Note:  
1. Basically, material can be applied in a single layer to achieve aesthetic restorations using Standard shades. For details, refer to the Clinical Hints.  
2. Material may be hard to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at room temperature.  
3. After dispensing, avoid too long exposure to ambient light. Ambient light can shorten the manipulation time.

**Clinical Hints**  
a. In the case of small cavities  
Restoring using a one shade layering technique. In most cases the use of one Standard shade alone will be sufficient. In cases where a higher degree of translucency is needed, one of the Outside special shades can be selected. See also Examples of Clinical Applications.  
b. In the case of large cavities  
In most cases a multi shade layering technique will give the best aesthetic results. To block out shine throughs from the oral cavity shade disc discolouration dentin, select an appropriate inside special shade and continue to build up with a Standard shade.  
To make a restoration more life like e.g. to copy age related changes in appearance, the final layer should consist of an Outside special shade. See also Examples of Clinical Applications and / or consult the Shade Companion Chart.  
2. Posterior cavities  
a. In the case of small cavities  
Restoring using a one shade layering technique. In most cases the use of one Standard shade alone will be sufficient. In cases where a higher translucency is needed, one of the Outside special shades can be selected. See also Examples of Clinical Applications.  
b. In the case of large cavities  
Place a flowable composite such as GC G-aenial Flo, GRADIA DIRECT Flo or GRADIA DIRECT LoFlo™ on the cavity floor. Then place a Standard shade. For optimal aesthetics use an Outside special shade as the final composite layer. See also Examples of clinical applications.

**Clinical Hints**  
a. In the case of small cavities  
Restoring using a one shade layering technique. In most cases the use of one Standard shade alone will be sufficient. In cases where a higher translucency is needed, one of the Outside special shades can be selected. See also Examples of Clinical Applications.  
b. In the case of large cavities  
Place a flowable composite such as GC G-aenial Flo, GRADIA DIRECT Flo or GRADIA DIRECT LoFlo™ on the cavity floor. Then place a Standard shade. For optimal aesthetics use an Outside special shade as the final composite layer. See also Examples of clinical applications.

**GC Fuji LINING® PASTE PAK, GC Fuji LINING® LC or GC Fuji IX GP** can also be used as a liner or base material. Follow the respective manufacturer's instructions for use.

**Examples of Clinical Applications (Clinical Hint No. 1, 2)**

**Shade combination chart for multiple layers in large cavities of anterior teeth (1-1)**

	A1	A2	A3	A3.5	A4	B1	B2	B3	C3
Inside special	BW	AO2	AO3	AO3.5	AO4	BW	AO2	AO3	AO4
Standard	A1	A2	A3	A3.5	A4	B1	B2	B3	C3
Outside special	JE	AE	AE	AE	AE	JE	JE	AE	AE

For details of shades, refer to the following section of SHADES.

5. Contouring before Light Curing  
Contour using standard techniques.

6. Light Curing  
Light cure G-aenial using a light curing unit (Fig. 3). Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

**G-aenial ANTERIOR: Irradiation Time and Effective Depth of Cure**

Irradiation time	Plasma arc (2000mW/cm²)	3 sec.	6 sec.
GC G-Light (1200mW/cm²)	10 sec.	20 sec.	40 sec.
TE, IE, JE, SE, CVE		3.0 mm	3.5 mm
A1, A2, B1, B2, XBW, BW, AE		2.5 mm	3.0 mm
A3, B3		2.0 mm	3.0 mm
A3.5, A4, C3, AO2, AO3, AO4, CV, CVD		1.5 mm	2.5 mm

**G-aenial POSTERIOR: Irradiation Time and Effective Depth of Cure**

Irradiation time	Plasma arc (2000mW/cm²)	3 sec.	6 sec.
GC G-Light (1200mW/cm²)	10 sec.	20 sec.	40 sec.
TE, IE, JE, SE, CVE		3.0 mm	3.5 mm
A1, A2, B1, B2, XBW, BW, AE		2.5 mm	3.0 mm
A3, B3		2.0 mm	3.0 mm
A3.5, A4, C3, AO2, AO3, AO4, CV, CVD		1.5 mm	2.5 mm

Note:  
1. Material should be placed and light cured in layers. For maximum layer thickness, please consult the tables.  
2. Lower light intensity may cause insufficient curing or discoloration of the material.  
7. Finishing and Polishing  
Finish and polish using diamond burs, polishing points and discs. To obtain a high gloss, polishing pastes can be used.

**SHADES**  
1. 22 shades for anteriors  
Standard shade : XBW (Extra Bleaching White), BW (Bleaching White), A1, A2, A3, A3.5, A4, B1, B2, B3, C3, CV (Cervical), CVD (Cervical Dark).  
Inside special shade : AO2, AO3, AO4  
Outside special shade : JE (Junior), AE (Adult), TE (Translucent)†, IE (Incisal), SE (Senior), CVE (Cervical).  
† TE shade is not radiopaque.  
2. 6 shades for posteriors  
Standard shade : P-A1, P-A2, P-A3, P-A3.5  
Outside special shade : P-JE (Junior), P-IE (Incisal)  
Note :  
A, B, C, AO shades are based on Vita® Shade.

**STORAGE**  
Store in a cool and dark place (4 – 25°C / 39.2 – 77.0°F) away from high temperatures or direct sunlight.

**Shelf life**  
G-aenial ANTERIOR - 3 years from date of manufacture  
G-aenial POSTERIOR - 3 years from date of manufacture

**PACKAGES**  
I. Quick Start Kit  
- 7 syringes (1 each in 7 anterior shades: A1, A2, A3, B2, AE, IE, JE) (2.7mL per syringe)  
- Shade guide  
2. REFILL  
- 1 syringe (in 28 shades) (2.7mL per syringe)  
Note:  
Weight per syringe : 4.7g for anterior syringe, 4.0g for TE shade, 5.5g for posterior syringe

II. Unitips  
1. Quick Start Kit  
(35 Unitips in 7 shades)  
- 35 tips (5 tips each in 7 anterior shades: A1, A2, A3, B2, AE, IE, JE) (0.16mL per tip)  
- Shade guide  
2. REFILL  
- 1 syringe (in 14 shades) (0.16mL per tip)  
(8 anterior shades: A1, A2, A3, A3.5, AO3, CV, IE, AE)  
(6 posterior shades: P-A1, P-A2, P-A3, P-A3.5, P-JE, P-IE)  
b. Pack of 10 tips (each in 14 shades) (0.16mL per tip)  
(14 anterior shades - XBW, BW, A4, B1, B2, B3, C3, AO2, AO4, CVD, TE, JE, SE, CVE)  
Note:  
Weight per Unitip : 0.28g for anterior tip, 0.24g for TE shade, 0.133g for posterior tip

III. Option  
1. Unitip APPLIER  
2. G-aenial SHADE GUIDE  
3. Mixing pad (No. 14B)

**CAUTION**  
1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water.  
2. In case of contact with eyes, flush immediately with water and seek medical attention.  
3. Take care to avoid ingestion of the material.  
4. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.  
5. For infection control reasons, Unitips are for single use only.  
6. Wear protective eye glasses during light curing.  
7. When polishing the polymerized material, use dust collector and wear a dust mask to avoid inhalation of cutting dust.  
8. Do not mix with other similar products.  
9. Avoid getting material on clothing.  
10. In case of contact with uninfected areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.  
11. Do not use G-aenial in combination with eugenol containing materials as eugenol may hinder G-aenial from setting.  
12. All shades except for the outside special shade TE are radiopaque.

Last revised : 06/2011

**CE 0086**

**MANUFACTURED BY**  
GC DENTAL PRODUCTS CORP.  
2-293, 2-chome, Higashi, Aizu-shi, Aizu-Wakekanai, Fukushima, Japan

**DISTRIBUTED BY**  
GC CORPORATION  
76-1 Haseguma-cho, Imai-shi, Tokyo 174-8686, Japan

**EU : GC EUROPE N.V.**  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL +32 16 74 10 00

**GC AMERICA INC.**  
3727 West 127th Street, Asip, IL 60803 U.S.A.  
TEL +1 312 697 0900

**GC ASIA DENTAL PTE. LTD.**  
10 Linyang Way, #06-27 Singapore 508274  
TEL +65 646 7588

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#### Kliniske tips

15. Konturering for lyshærdning

### G-anial LIGHT-CURED RADIOFASE COMPOSITE RESTORATIVE

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

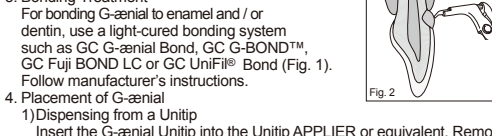
- G-anial ANTERIOR
- Direct restorative for Class II, III, IV, V cavities.
- Direct restorative for wedge-shaped defects and root surface cavities.
- G-anial POSTERIOR
- Direct restorative for veneers and distal onlays.
- Direct restorative for Class I and V cavities.

#### CONTRAINDICATIONS

- Do not use if the product may cause sensitivity in some people. If any and/or allergic reactions are experienced, discontinue the use of the product and refer to a physician.

#### DIRECTIONS FOR USE

- 1. Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate shade guide by referring to the G-anial shade guide.
- 2. Shade Preparation: Prepare cavity with standard techniques. Dry by gently blowing with free air.
- 3. For pulp capping, use calcium hydroxide.
- 4. Preparation: Prepare restorations using Standard shades. For details, refer to the Clinical Hints.
- 5. For bonding G-anial to enamel and/or dentin, use a light-cured bonding resin such as GC Clearfil Bond, GC G-BOND™, GC Fuji BOND LC or GC UniFil® Bond (Fig. 1).
- 6. Light Cure
- 7. Placement of G-anial
- 8. Finish
- 9. Polish

1. Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate shade guide by referring to the G-anial shade guide.

2. Shade Preparation: Prepare cavity with standard techniques. Dry by gently blowing with free air.

3. For pulp capping, use calcium hydroxide.

4. Preparation: Prepare restorations using Standard shades. For details, refer to the Clinical Hints.

5. For bonding G-anial to enamel and/or dentin, use a light-cured bonding resin such as GC Clearfil Bond, GC G-BOND™, GC Fuji BOND LC or GC UniFil® Bond (Fig. 1).

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