LC Block-Out Resin

Description and Indications:

Ultradent LC Block Out is a viscous light-cured, methacrylate-based, syringe delivered blue pigmented utility resin. It is designed for laboratory use such as for bleaching reservoirs on plaster models, for lab procedures requiring spacing or undercut block out, and to repair small fractures or voids in laboratory models.

Directions for Use:

NOTE: Stone /plaster models must be clean and dry before applying resin.

1. Remove Luer cap.
2. Securely attach working tip of choice (Black Mini tip).
3. Verify resin flow prior to applying.
4. Using palm grasp, express resin onto the model surface (Fig. 1)
5. **For Model Repair or Block-out (Figs. 2&3)**
 - a. Fill area no more than 3mm thick at a time.
 - b. Cure 20-40 seconds.
 - c. Add additional resin and cure as necessary.
 - d. Wipe off the oxygen inhibition layer.

For Custom Tray Reservoirs (Figs. 4&5)

- a. Apply 0.5mm of LC Block-Out to the desired surfaces.
- b. Keep resin approximately 1.5mm from the gingival margin.
- c. Cure resin 20-40 sec. per area.
- d. Wipe off the oxygen inhibition layer.
- e. Fabricate custom tray.


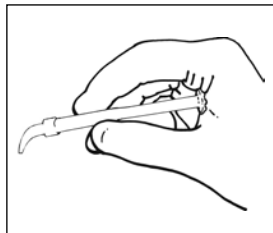
Fig. 1

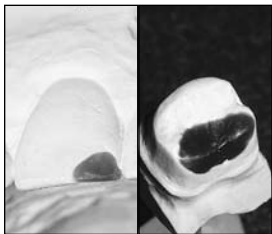Figs. 2 & 3

Fig. 4


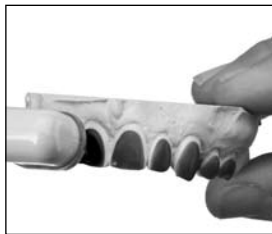
Fig. 5

Precautions and warnings:


1. Carefully read and understand all instructions before using.
2. Dispose of tip and syringe properly.
3. Methacrylate resins can be sensitizing. Wear gloves and avoid repeated contact of uncured dental resin with skin. If allergic reaction, dermatitis or rash develops, consult a physician.
4. Re-cap syringe between uses and store at room temperature.
5. Verify flow before applying. If resistance is met, replace tip and re-check. Use only recommended tips.
6. LC Block-out is not intended for intraoral use.
7. Keep product out of direct sun and heat.

NOTE: For MSDS and additional information about using LC Block-Out or other related products, please go to www.ultradent.com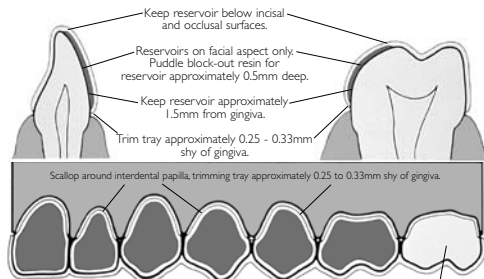

If one or more teeth are much darker than others (e.g., overly dark cuspids), build reservoirs on cuspids only to balance end whitening results.

Teeth of lower esthetic concern need not receive resin.

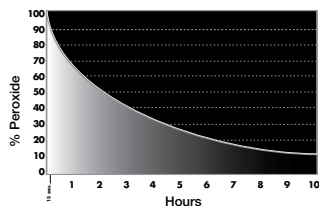Sustained Release Test: After dental examination and prophylaxis, Opalescence-filled customized tray (plastic membrane) was inserted. Small quantities of gel were removed at various intervals throughout the night for analysis of percentage of active ingredient.

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Ultradent Products, Inc.