DENTALON® plus

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GB Instructions for use

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SE Bruksanvisning

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NO Bruksanvisning

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Manufactured by:
Heraeus Kulzer GmbH
 Grüner Weg 11
 63450 Hanau (Germany)
 Made in Germany
 Distributed in USA/Canada exclusively by:
 Heraeus Kulzer, LLC, 300 Heraeus Way
 South Bend, IN 46614-2517, 1-800-431-1785
 Caution: Federal law restricts the sale
 by or on the order of a dentist.
 Importado e Distribuído por
 Heraeus Kulzer South América Ltda.
 CNPJ 48.708.010/0001-02
 Rua Cenzo Sbrighi, 27 – cj. 42
 São Paulo – SP – CEP 05036-010
 sac@kulzer-dental.com
 Resp. Técnica: Dra. Regiane Marton –
 CRO 70.705
 N° ANVISA: vide embalagem

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Status: 2013-09

DENTALON® plus

Selfcuring resin for temporary crowns and bridges, combination powder and liquid for the application performed by the dentist.

Colour	Colour code*	Colour code Biodent
L (light)	A3	17
M (medium)	A 3, C 3	23
D (dark)	A 3.5, A 4, B 3, B 4	41

Composition

Powder contains:
– polyethylmethacrylate/polymethylacrylate

Liquid contains:
– n-butylmethacrylate/urethanacrylate /ethylmethacrylate

Area of application

Fabrication of temporary crowns and bridges with the direct method, temporary crowns with confectioned crown forms.

Kind of application

• Mix powder and liquid in a ratio of 2 g:1.2 up to 1.5 ml.

• Place liquid in mixing cup. Dosing of the liquid can be achieved by using the dosing insert or by use of the enclosed dropper. To use the dropper, the dosing insert has to be removed from the bottle first. Pour powder and mix for 40 s with a glass spatula to a homogeneous mix. The resin is then pourable for 40 s. The plastic phase begins with 2 min and continues up to 5 min. Dentalon® plus has polymerized after 9.5 min and can thereafter be trimmed and polished immediately.

Mixing phase	Pouring phase (approx. 1 min)	Plastic phase (approx. 5 min)	Polymerisation (2.5 min)
40 s	2 min	7 min	9.5 min

Times mentioned are subject to ambient temperatures and can vary.

Handling

• Temporary crowns and bridges
Prepare situation impression with alginate or silicone prior to preparation. Keep impression until termination of preparation (store alginate impression wet). Prepare teeth, thereafter insulate cores thinly with vaseline or silicone grease. Mix Dentalon® plus and pour during pouring phase into impression. With temporary bridges remove as much impression material between pontics to be able to model a functional bridge unit. As soon as a good plastic consistency is reached, insert impression into mouth. During the curing phase is it possible to remove impression and remove resin excess.

• Select crown cap after preparation and cut cervically. Thereafter insulate core with vaseline or silicone grease in a thin layer. Mix Dentalon® plus. Fill up crown cap during pouring phase and insert during plastic phase. Remove crown from core during curing phase and complete polymerization.

Instructions for use GB

Dentalon® plus is treated with coarse and middle fine staggered diamond burs. Treat material without pressure and polish to high gloss with a rag wheel and conventional polishing paste.

• Crown and bridge temporaries are luted in with conventional temporary luting cements.

Further precautions

• Dentalon® plus should not be used if allergies resulting from components of the system are known.

• Systemic side effects are not known. Single cases of allergic type IV reactions are mentioned.

• Discoloration can happen if in contact with cationic mouthwashes as well as plaque releators and chlorhexidine.

• Do not use material after expiry date. Do not store above 25°C (77°F), avoid direct exposure to sun. Close container immediately after use. Keep out of reach of children.

• Irritating; product contains ethyl methacrylate, urethandiacyrlate, butylmethacrylate. Inflamable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact. Keep away from sources of ignition – No smoking. Avoid contact with skin. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable gloves.

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* Registered trademark of Vita-Zahnfabrik, Bad Säckingen